



Silensor-sl anti-snoring device

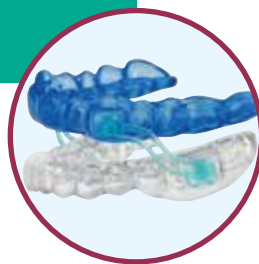

How to replace **Silensor-sl** connectors

1. The upper splint

5. The upper anchor

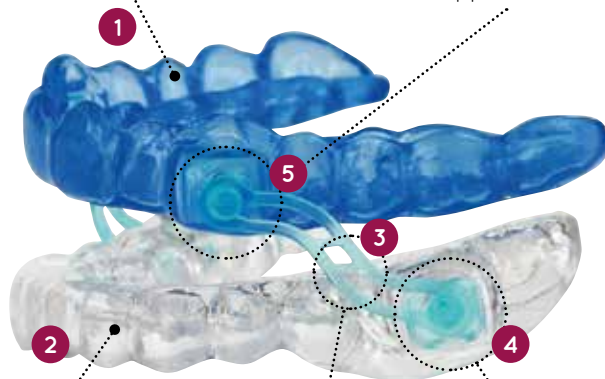

2. The lower
splint

3. The double
S shaped
connector

4. The lower
anchor

Silensor-sl parts guide



Upper splint



Connector



Anchor




Lower splint

How to replace **Silensor-sl** connectors

HOW TO REMOVE YOUR CONNECTORS




Step 1
Slide the lower anchor forward.

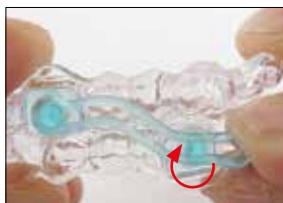


Step 2
Twist the connector off of the anchor. Repeat this process until both connectors are removed.

SELECT YOUR CONNECTORS

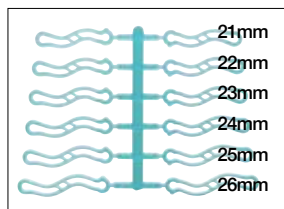


1. Select your connector from the tree.


A shorter connector provides more protrusion.
A longer connector provides less protrusion.

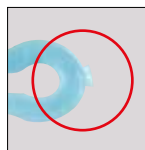

2. Cut the connector cleanly off of the tree ensuring you remove the connection point.

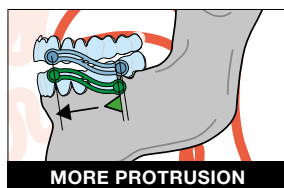

MORE PROTRUSION

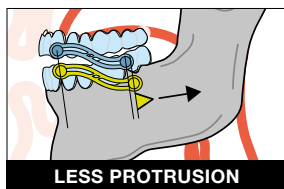

LESS PROTRUSION

Important

Please ensure you follow the advice and instructions of your dentist

HOW TO REPLACE YOUR CONNECTORS



Start on the upper anchor. Take the connector (A: *Nose to nose*) and using your index finger apply pressure to the back of the anchor. Then simply push and twist the new connector on. Repeat process on the lower anchor (B)

Nose to nose

